


# Liko™ Multirall™ 200

An overhead lift for flexible needs

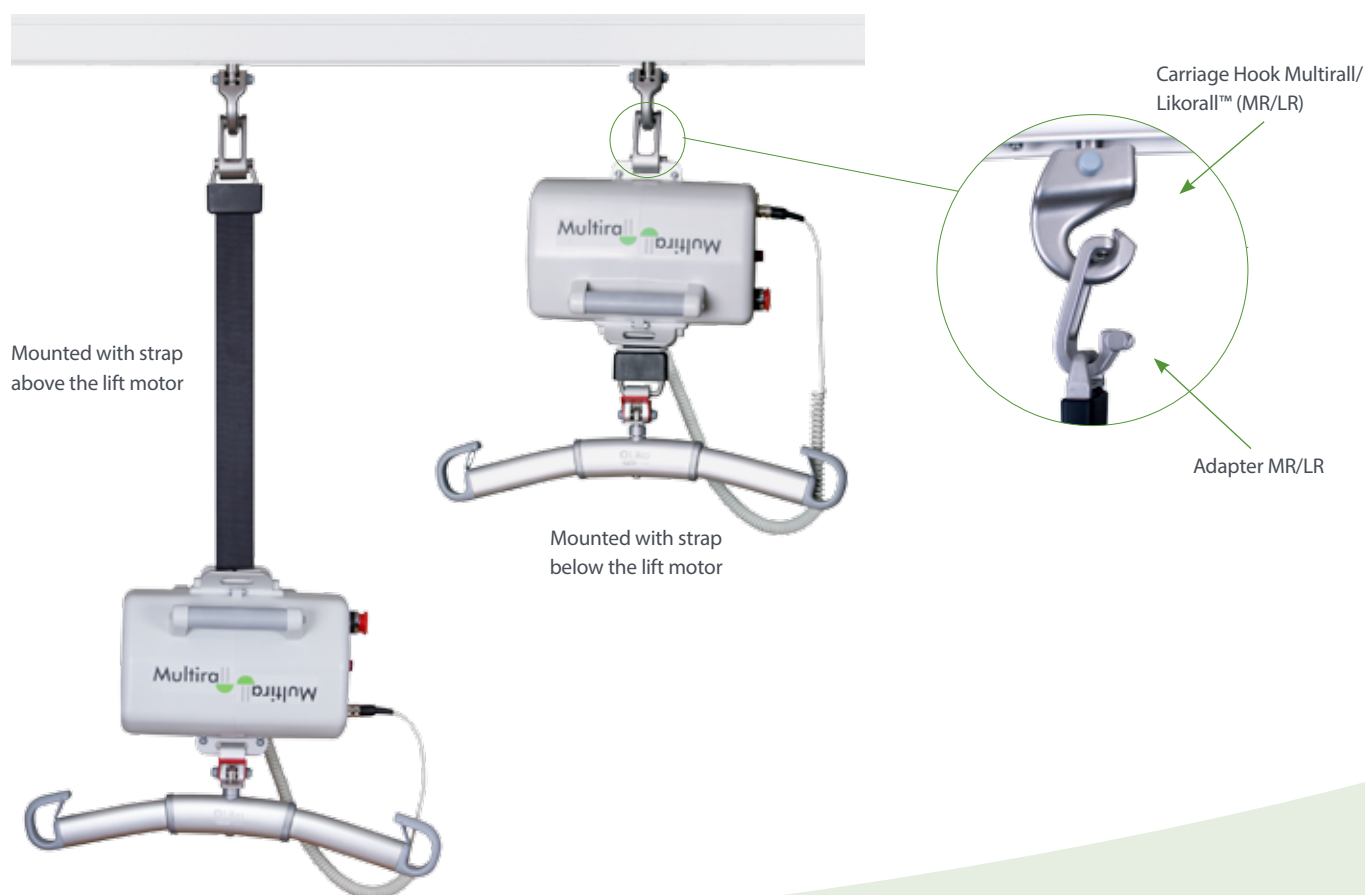

In care environments such as hospitals and nursing homes lifting needs may vary, depending on the mobility of the patient. A portable overhead lift can provide a good alternative to a mobile lift. Multirall offers a cost-effective portable lifting solution. It can also be used to transfer a patient between rooms in a sitting position, easily moving from one rail system to another with the use of a room-to-room strap.

## Features and benefits

### Can be mounted two ways:

1. With the strap below the lift motor, for more stationary use
2. With the strap above the lift motor, for more mobile use

The **Adapter MR/LR** offers a safe and easy-to-use interface between the Multirall and the Carriage Hook MR/LR, regardless of which way it is mounted

**Easy to move** with the optional Extension Arm Adapter MR, no tools needed

Large handles on both sides of the motor facilitate **easy transport**

Can be used with all Liko rail systems, including FreeSpan™, the free-standing option

**An extensive assortment of accessories** is available including slings, slingbars, scales and the Multirall Trolley

# Liko Multirall 200

## Technical Specifications

Safe Working Load	200 kg
Lift motor weight	8.7 kg
Slingbar options	Mini slingbar 220, Universal slingbar 350, Universal 450, Universal 600, Universal TwinBar 670, Crossbar
Slingbar connections	Quick-Release
Lift motor safety class	IP 30
Emergency lowering	manual and electrical
Lifting speed	60 mm / sec
Lifting interval	1600 mm

Sound level	62.2 dB(A)
Battery	2 x 12V DC (2.4 Ah - 2.6 Ah) valve-regulated lead-acid gel-type batteries
Battery charger	wall charging
Hand control	wired
Protection class hand control	IP 43
Product number	3130001

Hill-Rom is a leading global medical technology company with more than 10,000 employees worldwide. We partner with healthcare providers in more than 100 countries, across all care settings, by focusing on patient care solutions that improve clinical and economic outcomes in five core areas: **Advancing Mobility, Wound Care and Prevention, Patient Monitoring and Diagnostics, Surgical Safety and Efficiency and Respiratory Health.** The people, products and programs of Hill-Rom work towards one mission: **Every day, around the world, we enhance outcomes for patients and their caregivers.**

Not all products/options are available in all countries. For further information about our products and services, please contact your local Hill-Rom representative or visit our webpage:

[www.hill-rom.com](http://www.hill-rom.com)

Liko Multirall 200 is intended for use in all common lift and transfer situations, for example, between bed/wheelchair, to/ from floor, toilet visits, gait training, and for horizontal lifts with stretchers.

Class: I

Manufacturer: Liko AB, Nedre vägen 100, SE-975 92 Luleå, Sweden

This medical device is a regulated health product which, pursuant to such regulation bears a CE mark. Hill-Rom recommends that you carefully read the detailed instructions for safe and proper use included in the documents accompanying the medical devices. The personnel of healthcare establishments are responsible for the proper use and maintenance of these medical devices.

Hill-Rom reserves the right to make changes without notice in design, specifications and models. The only warranty Hill-Rom makes is the express written warranty extended on the sale or rental of its products.

©2017 Hill-Rom Services, Inc. ALL RIGHTS RESERVED.

Doc. No: 5EN183328-01, 12 May 2017

Enhancing outcomes for  
patients and their caregivers:

**Hill-Rom**