

Golvo™ Lift



The complete mobile lift

Based on a world-patented lifting principle, the Golvo™ lift is unique in terms of flexibility and versatility.

Mechanical emergency lowering

Patented emergency-lowering function for quick and easy lowering.

Handcontrol

Easy-to-use handcontrol with magnetic mount.


Built-in charger

The built-in charger always stays in place on the lift. Simply charge the batteries by plugging the charger cable into a wall outlet.

Electrical emergency lowering and raising

In the event that the handcontrol for some reason stops working.



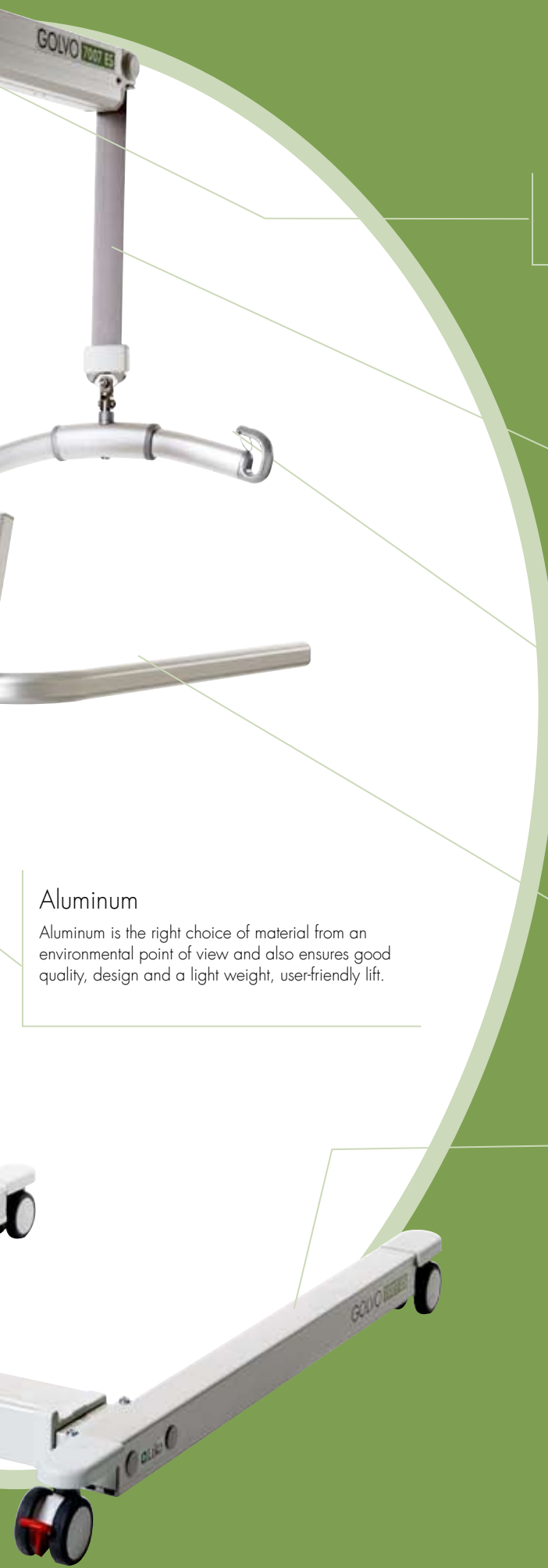

Vertical lifting

The easiest and most space efficient lifting motion.

Flexible lift strap

Adapt the lift to suit the person being lifted.

Slingbar with safety latch

Prevents the slingbar from unhooking unintentionally, and makes the lift safe and secure.

Retractable arm rests

For both the patient and the caregiver – added reassurance.

Parallel base-width adjustment

Golvo's base can be widened and narrowed with the unique parallel adjustment, enabling access around beds or wide chairs etc.

Aluminum

Aluminum is the right choice of material from an environmental point of view and also ensures good quality, design and a light weight, user-friendly lift.

Unsurpassed Flexibility

The "chameleon" of patient lifts

Unique in design and function, the Golvo lift has been developed with an insight into the great variety of lifting needs that arise in the health care sectors. The Golvo lift's flexibility and versatility, together with Liko's comprehensive range of functional slings and accessories, make it the ideal lift for most situations.

↑ Safety and rehabilitation

Together with the Liko MasterVest™ or Liko Lift Pants slings, the Golvo lift offers a popular means of risk-free gait training. Armrests give support and reassurance. The MasterVest sling can also be used for visits to the toilet.

← Lifting children

The Golvo lift's wide lifting range and versatility make it the ideal aid for lifting and stimulating children and young people. It can lift to high tables and lower right down to the floor. The Golvo lift features a wide range of lifting accessories specially adapted for children.

← Parallel base-width adjustment

The Golvo lift's base can be widened and narrowed with the unique parallel adjustment. This allows the patient to be lifted into a wheelchair from the perpendicular and still achieve the correct sitting position.

← Lift with ease from the floor

Lifting a patient from the floor is surprisingly easy. The open base provides plenty of room, and the flexible slingbar suspension with one strap enables you to adapt the lift to suit each patient. The Liko Original High Back Sling gives the patient the best support.





Life is fantastic!

Everyone should have the right to live life
in the best way possible

Easy getting to the toilet



When the need arises

Lifting in the toilet area is often difficult and challenging. The Golvo lift is versatile, even in hygiene areas where space is limited. There are slings for every situation. As always, the most appropriate sling is selected on the basis of the patient's condition and clothing.



With Liko Original Sling (Mod. 10)

Often, toileting can be managed easily with one of Liko's basic sling models, e.g., Original or Universal.

The patient can be transferred to the toilet with a mobile hygiene chair.



With Liko Hygiene Sling (Mod. 40, 45)

Patients with good upper body strength can use Liko Hygiene Sling. Liko Hygiene Sling has a generous opening around the bottom that facilitates undressing during the lift.

With Liko Hygiene Vest (Mod. 50, 55) →

Liko's unique Hygiene Vest solves most toileting problems. It provides unique advantages in terms of reassurance to both patient and caregiver. Many patients appreciate its stable, upright sitting position.

Hygiene Vests are available with head supports.



Liko MasterVest™ Sling (Mod. 60, 64)

The Golvo lift's high lifting height makes it possible to raise a patient to the standing position. This facilitates undressing and dressing while using the toilet. Anyone capable of supporting their weight while standing can be lifted to a standing position with the Golvo lift and Liko MasterVest sling.



Respect and dignity

are important
when you are lifting
another
human being





Liko OctoStretch

Safe, secure lifts require more than just technology



Lateral transfer stretcher
with Stretch Leveller



LikoScale with Quick-release
adapter

Liko Stretchers

Rely on the Golvo™ lift in every situation

Lifting a person from a supine or prone position places high demands on the lift and lifting accessories. The Golvo lift's stability is reassuring. The lift is achieved safely and securely, mainly due to the unique parallel adjustment, plus the lifting motion is vertical, keeping the center of gravity in the middle of the lift. Many lifting accessories are available for the Golvo lift and the special needs of the acute limit. For example:

← Liko OctoStretch™ Device

The OctoStretch device, pictured here with a full lifting sheet, is an easy-to-use and much-appreciated aid in operating rooms and intensive-care units. Several types of lifting sheets are available for the OctoStretch device.

LikoStretch™ 600 IC Device →

A horizontal lift for the most difficult and delicate patients.

The lift strap is passed in under the patient with a pliable spatula, avoiding the need to turn the patient. Liko's unique strap-lock secures the patient in place, so there is no need to alter his or her position during lifting.

← LikoScale™ Device

A user-friendly scale is available as an optional accessory for use with Liko's lifts. Weighing patients in the lift is practical and convenient. The patient sits comfortably in the sling whilst the caregiver reads the weight, clearly visible on a display at eye level. Mounted with the quick-release adapter, the scale can be easily moved when necessary.

← Liko Stretch Leveller

With the Stretch Leveller mounted on the horizontal lift, the balance can be adjusted quickly and easy, avoiding tipping of the patient.

FlexoStretch™ System

Designed for the heaviest lifting situations with the patient in a horizontal position. Lift sheet or lift straps are optional accessories.

A Stretch Leveller is part of FlexoStretch system.



LikoStretch 600 IC Device



FlexoStretch System

Down to the smallest detail

The Golvo™ lift is the most versatile mobile lift on the market.

Together with Liko's lifting accessories, the Golvo lift is ideal in most lifting situations.



The mobile overhead lift

The Golvo lift uses the principles of an overhead lift. This provides unique flexibility.



Vertical lifting motion

Gives maximum stability, since the center of gravity is always directly in the middle of the lift.



Parallel base-width adjustment

The Golvo lift's base width has a unique parallel adjustment. This facilitates access in many different lifting environments, to and from the bed, toilet, x-ray table, etc.



Adjustable lifting range

When lifting from the floor, the slingbar can be altered to the desired level. Golvo can reach all the way.



Armrests

The retractable armrests give reassurance and stability for the user when gait training, toileting etc.



Mechanical emergency lowering

Patented emergency-lowering function for quick, trouble-free lowering.

Smart accessories

Quick-release hook and Q-link

Liko has developed a system that enables quick and easy exchange of lifting accessories - without the need for tools. Change from a slingbar to a stretcher, or connect a scale in one simple operation. Pictured, the LikoScale™ device with quick-release hook TDM and Q-link on the lift strap.



Extra charger station

With a wall-mounted charger station and an extra battery, reserve power is always available.

Dependable electronics

Handcontrol

Two speeds for even control and base width adjustment.



Built-in charger

The batteries and charger are contained in the one control box attached to the lift. Just plug the charger cable into a wall outlet. Overcharge limiter terminates charging when the batteries are fully charged.

Display for charge status

A clear audible and visual indication on the display tells you when the batteries need to be charged.

Electrical emergency lowering and raising

If the handcontrol for some reason stops working, The Golvo lift can be operated directly from the control box. The raising function allows you to make urgent lifts.

Electrical emergency stop

Cut the power supply quickly by simply pressing a button.



Technical data

- Lifting capacity: 440 lbs. (200 kg)
- Lifting speed: Two speeds; 1.9 inch./s (4.8 cm/s) and 1.3 inch./s (3.2 cm/s), respectively, without load
- Batteries: 2 x 12V 2.9 Ah gelyte batteries (valve regulated, sealed, lead accumulator)
- Battery charger: Built-in charger for 100-240 V, 50-60 Hz, max 400 mA
- Material: Anodized aluminum



Safety latch

A simple, ingenious device that prevents the sling from unhooking unintentionally.



Colors and sizes

Color coded sling size chart added to co-ordinate with the Liko range of slings.



Sit-to-stand

Sit-to-stand accessories can be mounted on the Golvo lift without getting in the way during normal lifts. Easy installation; no tools required.



The Golvo lift 7000 ES/7007 ES is a Class 1 medical-technical product that complies with the medical-technical directive MDD 93/42/EEC and the requirements of EN ISO 60601-1. Made in Sweden

Lifting accessories

Together with Liko's comprehensive range of accessories, the Golvo lift offers solutions to virtually all lifting problems in the care sector. Contact your Liko representative or visit www.liko.com for further information and advice.



Global Leadership in Advanced Design and Application Excellence

Lifting a fellow human being requires both high quality technical devices and clinical expertise. Liko specializes in lifting and transferring people whose mobility is impaired. Our specialization enables us to remain at the forefront of health care development.

For 30 years, Liko has specialized successfully in the design and manufacture of safe, easy-to-use assistive devices for lifting and transferring physically disabled persons. Our success is based on a unique combination of sophisticated technical skills and clinical know-how.

Our product range includes mobile lifts and overhead lift systems, slings, and accessories. Liko is represented by its own subsidiaries and agents in more than 30 countries worldwide. Please don't hesitate to contact us or one of our agents for further information and support.

Liko is quality certified according to ISO 9001 and environmentally certified according to ISO 14001. Liko lifts comply with ISO 10535



www.liko.com

Manufactured by

Liko AB
Nedre vägen 100
SE-975 92 Luleå
Sweden
info@liko.se

National Distributor in USA and Canada:

Liko
122 Grove Street
Franklin, MA 02038
Tel: 508-553-3993, 1-888-545-6671
Fax: 508-528-6642
info@liko.com

